



# Step-by-Step Clean Up Guide

Instructions for use for the enzymatic and surfactant Clean Up Pouches. This simple bedside kit boasts a quick and easy post-procedure manual clean.

1



Firmly tap the pouch on a hard surface to bring the liquid to the bottom of the pouch.

2



Find the tear notch, tear across and downwards to easily open the pouch.

3



Pour in potable water until it reaches the specified fill line (approx. half fill pouch). Reseal for subsequent use.

4



Once procedure is completed, remove sponge and squeeze to remove excess fluid.

5



Thoroughly wipe the endoscope down using the sponge.

6



Submerge the tip of the endoscope into the detergent and draw through into the working channels.

7



Finally, wipe down the handle using the sponge and prepare the endoscope for transport.