



PARASURE

ParaSure CEI

Cleaning Efficacy Indicator

- For monitoring the cleaning efficiency of washer- disinfectors
- Test soil simulation of human blood in accordance with ISO 15883
- Diagnostic - Easy to use, Easy to read
- Stainless steel - Non toxic
- Routine monitor/validation tool
- For use in Washer Disinfectors and Ultrasonic machines



ParaSure CEI (Cleaning Efficacy Indicator) cleaning efficiency check is designed for the validation and the routine monitoring of the cleaning efficacy of cleaning processes in washer disinfectors for surgical instruments.

ParaSure CEI a special cleaning indicator made of stainless steel, which is contaminated with a quantitative measure of test soil (blood and additives), according to ISO 15883. This product is easy to use, easy to read and can be disposed of with domestic waste.

Order Details

PS1600

Cleaning Efficacy Indicators

Box of 50

Registered Address:

ParaSure Limited Units B10/11 Oakendene Estate Bolney Rd Cowfold West Sussex RH13 8AZ England

+44 (0)1403 864777

Ref: ParaSure Tech Sheets 08/12 Rev.B



WWW.PARASURE.CO.UK



WWW.PARASUREORGANICS.COM



PARASURE

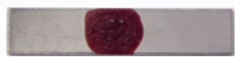
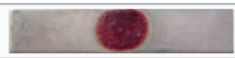


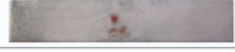
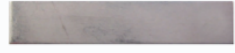
ParaSure CEI

Cleaning Efficacy Indicator

TECHNICAL SPECIFICATIONS

Name:	ParaSure CEI (Cleaning Efficacy Indicator)
Classification:	ISO 15883
Low Storage conditions/RH:	+18°C - RH 35%
High Storage conditions/RH:	+25°C - RH 70%
Lot no:	yes
Prod date:	yes
Exp date:	yes
Correct symbols:	In accordance with ISO 15883
Indicator:	Cleaning Efficacy Indicator
Disp/single use:	yes
Extra information: Non Toxic Shelf Life	yes 18 months

Assessment Aid

Result	Assessment	Action needed
	Reference indicator: Spot of test soil completely visible	
INSUFFICIENT CLEANING PROCESS:		
	Spot of test soil completely visible	Do not release batch!
	Spot of test soil visible	Check load, temperature, dosage and time.
	Spot of test soil visible	Repeat test.
	Spot of test soil slightly visible	In case of repeated insufficient cleaning process: Call service technician.
SUCCESSFUL CLEANING PROCESS:		
	Spot of test soil completely removed	Release batch. No further action needed.

For further information about any of our products, please visit:
www.parasure.co.uk



WWW.PARASURE.CO.UK

Registered Address:

ParaSure Limited Units B10/11 Oakendene
Estate Bolney Rd Cowfold West Sussex RH13
8AZ England

+44 (0)1403 864777



WWW.PARASUREORGANICS.COM